
# Liskosil

# Liskosil-s Liskosil-m Liskosil-l

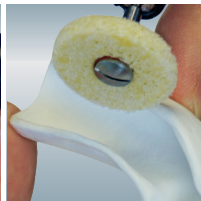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## EN: Application

**Dental field:** all thermoforming materials, printed and milled plastics, also metals and silicones.

**Orthopedic field:** orthoses, ball protection, wedge between the toes etc.

Liskosil is a fine-abrasive polishing disc with an enormous service life, which not only allows fine-grained removal of material, but also removes larger quantities of material by increasing the contact pressure. No matter whether it is coarse or fine sanding, a fine, slightly mat surface is always left, which in many cases does not require a high gloss polish.

Liskosil-l is particularly suitable for large-area processing.

Larger quantities of material can be removed without leaving an edgy surface.

Liskosil-m allows the same result as Liskosil-l in narrow areas, e.g. in the papilla area of a splint.

Liskosil-s, the small diameter allows the processing of occlusal interferences and even of the inner surface of a splint.

**Ideal speed range for Liskosil, 5000 - 10000 rpm, wear protective goggles.**

Delivery form (large packs on request):

**223 220** Liskosil-s, polishing discs, 6 pieces and 1 mandrel

**223 230** Liskosil-m, polishing discs, 6 pieces and 1 mandrel

**223 240** Liskosil-l, polishing discs, 6 pieces and 1 mandrel

**223 250** Liskosil polishing discs, start set, one polishing disc each, -s, -m, -l and 3 mandrels

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